

Don't wait until your hearing aids stop working to take charge of your hearing aid maintenance.

It is now easy to protect your hearing aids from moisture and earwax, the two most common cause of hearing aid failure.

Your **Hearing Healthcare Professional** can show you how easy it for you to maintain your hearing aid to peak performance.

Moisture and earwax are the two most common cause of hearing aid failure.

If you have difficulty handling your hearing aids, ask a family member or friend that is willing to help. Everyone benefits when hearing aids are functioning at peak performance.

Easy steps for cleaning a hearing aid



Power-Vac was designed after observing hundreds of hearing aids being sent in for repair for clogged receiver (speaker) and or microphone ports that could have been cleared by the wearer. The Power-Vac is completely portable, operates on 2 AA batteries or DC converter.

a. The entire hearing aid should be brushed to remove any loose wax or debris. The brush and or wax-loop on the Combination Cleaning Tool can loosen or remove wax from the receiver port area. The vent, earmold or open fit tube should be cleared with the open fit cleaning tool.

b. The hearing aid (or earmold) is then placed on the standard suction tube to further remove loose wax and debris from the hearing aid with a combination of suction and vibration. The standard tube can also be used to "vacuum" the entire surface of the hearing aid.

c. If needed, your **Hearing Healthcare Provider** may show you how to use a cone shaped suction tips to clear receiver (speaker) and

microphone ports by matching the diameter of the tubing with the cone shaped suction tip. **Do not insert any object into hearing aids with wax guard filters.**

d. Wipe the entire surface of the hearing aid to remove any loose debris prior to use.

Protection from Moisture Damage



Easy to Use

Universal Hearing Aid Dryer™

Prevents costly breakdowns and maintain performance. Safely removes harmful moisture and helps dry wax for easier removal. Designed for all style of hearing aids including Open Fit.

At The End Of The Day

a. Slide the switch to the ON position the Blue indicator lights should now be bright.

b. Remove the batteries from the hearing aid(s). Place the hearing aid(s) with the canal aimed down in the center of the **BLACK** heating area leaving the battery doors open.

c. Allow the hearing(s) aid to dry overnight.

In The Morning

d. Slide the switch to the OFF position the Blue indicator lights should now be dim. **ALLOW THE HEARING AID(S) TO COOL FOR 5 MINUTES BEFORE USING.**

e. Use a cleaning brush to remove any loose debris and wipe the hearing aids with a dry tissue.

Regularly scheduled check-ups with your Hearing Healthcare Professional are recommended.